




Field Installation and Usage Instructions

Part No. TMA11-15, TMA160-15, and TMA181-15

Seat Surgical Bar Assembly

For All Model TMM Chairs

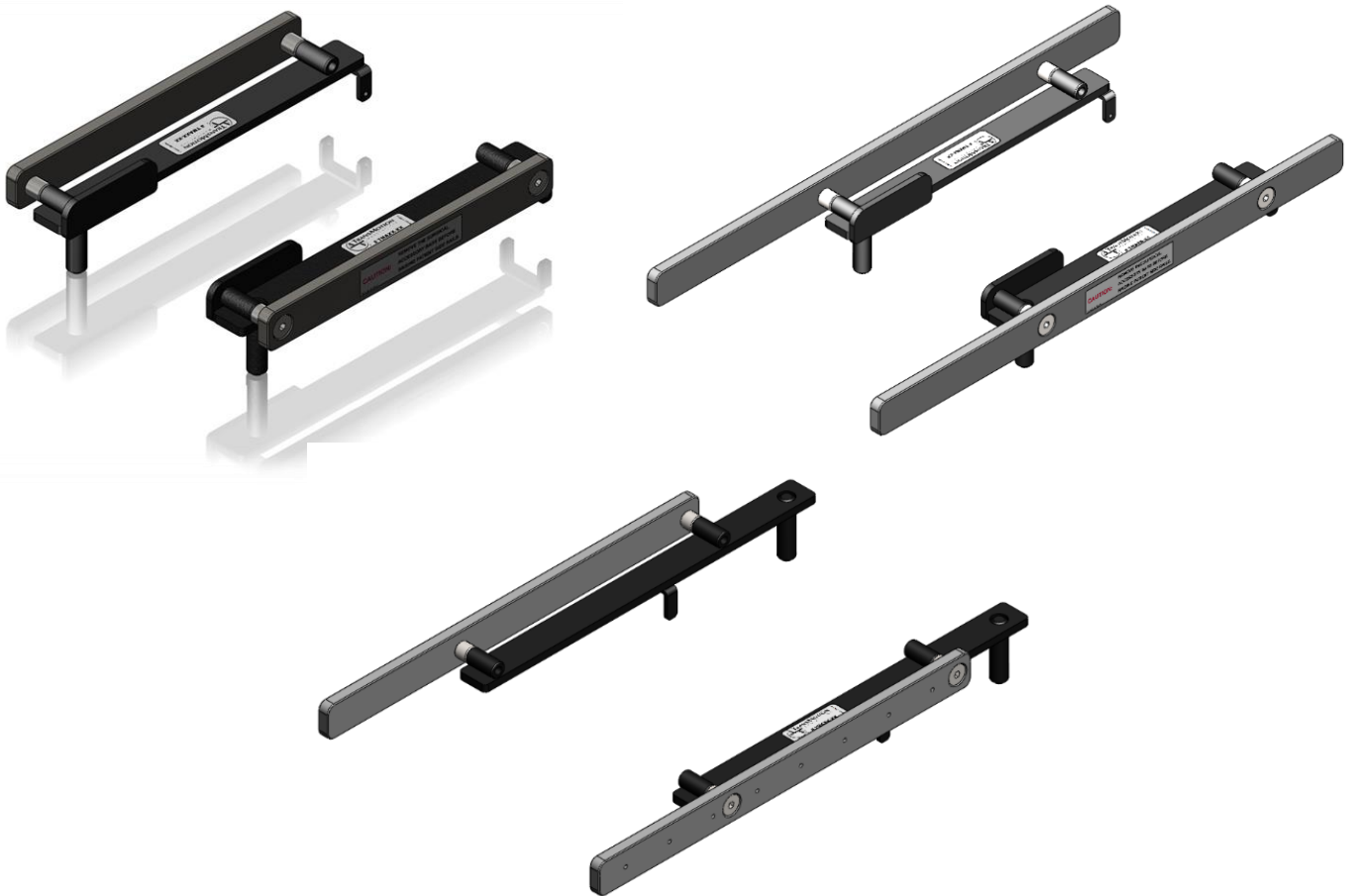

5516 S.W. 1st Lane – Ocala, FL 34474-9307
Phone: 1-800-237-3377 - Fax: 1-352-854-9544
www.transmotionmedical.com

Document No.: IM TMA(11,160,181)-15
Revision: D

TABLE OF CONTENTS

IMPORTANT NOTES ABOUT THIS MANUAL.....	3
SPECIAL NOTES - SIGNAL WORDS	3
TOOLS REQUIRED.....	4
PARTS INCLUDED IN TMA11-15, TMA160-15, OR TMA181-15 KIT	4
INSTALLING SURGICAL BAR ASSEMBLY.....	5
USING SURGICAL BAR ASSEMBLY	6
CAUTIONARY USAGE INSTRUCTIONS.....	7
PREVENTIVE MAINTENANCE	7
CLEANING INSTRUCTIONS	8

IMPORTANT NOTES ABOUT THIS MANUAL

This Field Installation and Usage Instruction document is designed to assist you with installation of the TransMotion Medical Product. Carefully read this document thoroughly before using the equipment or doing service/maintenance on it. If you are unable to understand the WARNINGS and CAUTIONS and instructions, contact TransMotion Medical customer service before attempting to operate or service the equipment otherwise injury or damage may result.

To assure safe operation of this device, it is essential that methods and procedures be established for educating and training staff on the safe and effective operation of the product.

TransMotion Medical reserves the right to change specifications without notice.

The information contained in this document is subject to change without notice.

Specifications listed are nominal and may vary slightly from unit to unit due to tolerance variations.

Photographs and illustrations contained within this document may not depict exactly the model you have. This document can cover many closely related models.

SPECIAL NOTES - SIGNAL WORDS

Signal words are used in this manual and apply to hazards which could result in injury or property damage. The following is a definition of those signal words as used in this document:



Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.



Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices or potential property damage hazards.



Provides important information, makes special instructions clearer, or provides service personnel information to make maintenance easier.

TOOLS REQUIRED

No tools required to install this accessory.

PARTS INCLUDED IN TMA11-15, TMA160-15, OR TMA181-15 KIT


Verify that you have all the parts shown. If not, contact TMM Service at 1-800-237-3377.

(2) Surgical Bar Assemblies (One of the sets shown to the right)




INSTALLING SURGICAL BAR ASSEMBLY

Rotate side rails into lowered position until rail is firmly held by catch.

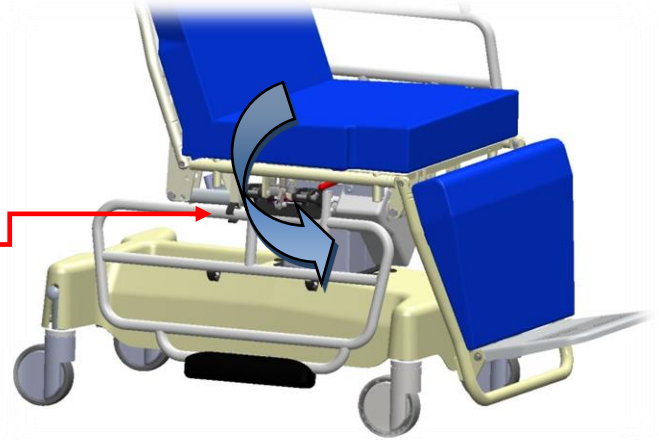

TMA11-15 and TMA160-15
Locate front IV wells on each side of unit

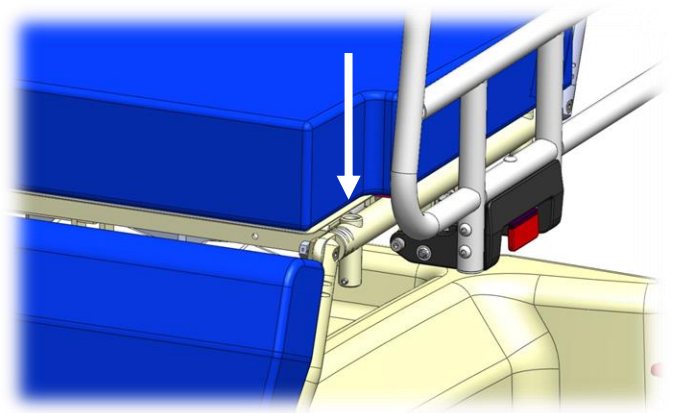


Align IV well legs in surgical bar assembly with IV wells in TMM chair seat section and insert.



NOTICE

Ensure rear bracket of surgical bar assembly secure around seat weldment

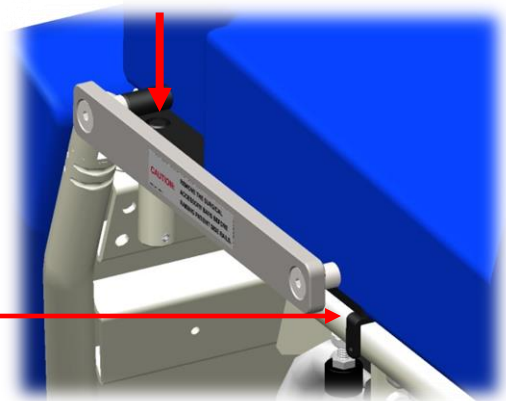


TMA181-15

Locate rear IV wells on each side of unit. Align IV well legs in surgical bar mounting assemblies with rear IV wells in TMM chair seat section and insert.



NOTICE

Ensure front bracket of surgical bar mounting assemblies secure around seat weldment.

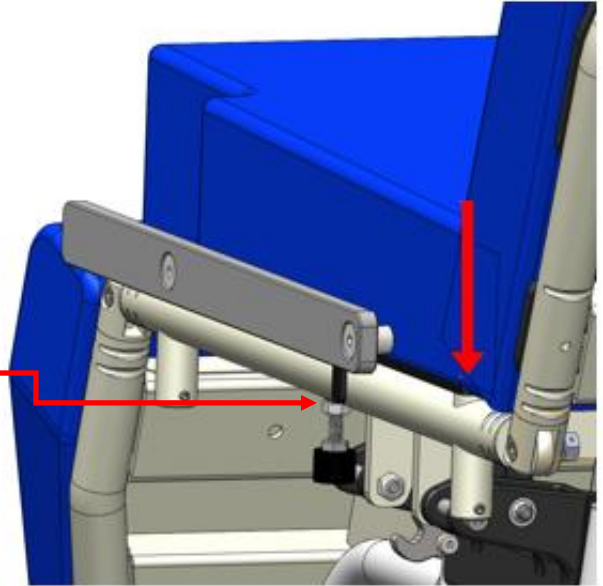


USING SURGICAL BAR ASSEMBLY

Use surgical bar assembly by securely mounting equipment or tools as needed.



CAUTION

To prevent personal injury to medical personnel and/or patient, remove surgical bars when not in use or ensure patient and medical personnel's hands and arms are clear of the surgical bar before raising the patient side rails. Surgical bar assemblies can be left on the chair when the heel stirrups are not in use but leaving them on the device could result in a possible pinch point between the side rails and surgical bars.

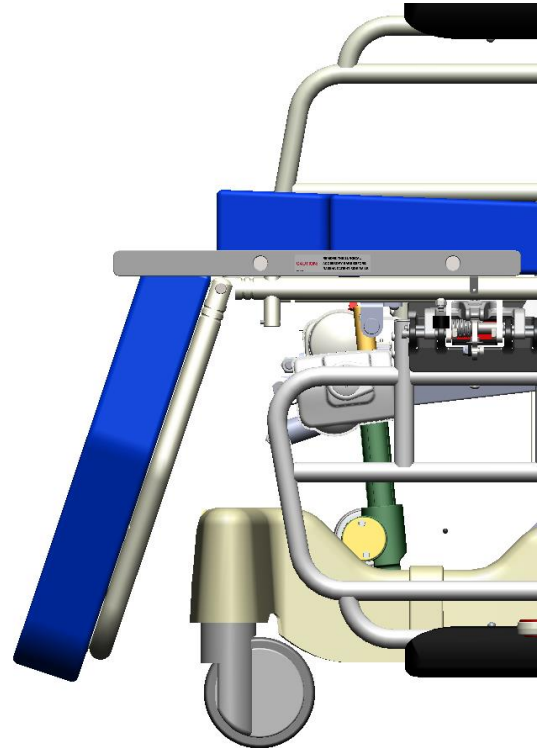

CAUTIONARY USAGE INSTRUCTIONS

When using this accessory, please follow these cautionary usage instructions:

CAUTION TO PREVENT PERSONAL INJURY TO MEDICAL PERSONNEL AND / OR PATIENT, ENSURE SURGICAL BAR ASSEMBLY IS PROPERLY INSTALLED AND SECURELY ENGAGED PRIOR TO TRANSPORTING PATIENT / CHAIR, AND PRIOR TO USE.

CAUTION TO PREVENT PERSONAL INJURY TO PATIENT, ENSURE SURGICAL BAR ASSEMBLY IS REMOVED BEFORE A PATIENT ENTERS OR EXITS THE STRETCHER-CHAIR OR ASSIST PATIENT AS THEY ENTER OR EXIT THE STRETHCER-CHAIR.

CAUTION THE MAX WEIGHT OF EACH SEAT SECTION SURGICAL RAIL IS 100 POUNDS. TO PREVENT PERSONAL INJURY TO PATIENT, DO NOT MOUNT EQUIPMENT WEIGHING MORE THAN 100 POUNDS TO THE SURGICAL RAILS. WEIGHT OF PATIENT PLUS WEIGHT OF ANY EQUIPMENT MOUNTED TO THE CHAIR SHOULD NOT EXCEED THE WEIGHT CAPACITY OF TRANSMOTION MEDICAL STRETCHER-CHAIR.

CAUTION ONLY EQUIPMENT APPROVED BY TRANSMOTION MEDICAL IS TO BE MOUNTED ON THESE SURGICAL RAILS. TRANSMOTION MEDICAL IS NOT RESPONSIBLE FOR DAMAGE, AND ASSUMES NO LIABILITY, CAUSED BY THE USE OF UNAPPROVED EQUIPMENT. APPROVED MEDICAL EQUIPMENT INCLUDES ANY TOOLS, INSTRUMENTS, OR SCOPES THAT ARE COMPATIBLE WITH A .365" THICK BY 1.125" WIDE SURGICAL RAIL.

PREVENTIVE MAINTENANCE

To ensure proper operation, this accessory should be inspected and maintained as described below.

COMPONENT	FREQUENCY	PROCEDURE	TOOL	COMMENT
Screws	Every three months	Tighten screws	5/16 Hex key	-
Surgical Bar	Every three months	Visually inspect surgical bar	-	If bar appears bent or damaged contact TransMotion Medical for replacement

CLEANING INSTRUCTIONS



STEAM AND/OR PRESSURE CLEANING CHAIR WILL VOID WARRANTY

COMPONENT	FREQUENCY	PROCEDURE	CLEANING AGENT *	COMMENT
All Rails and Painted Metal	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for chipped paint/chrome
Base Cover	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for peeling or missing labels
Actuators and Battery Pack	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
Pendants	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
All Other Surfaces	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
TransMotion Medical Accessories	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
3rd Party Accessories	As required/in accordance with facility policies	Consult manual provided by accessory manufacturer.		

* Use of Facility approved detergents and disinfectants must comply with the instructions provided by the manufacturer(s) of those products.

Upholstery Care & Cleaning Instructions

Proper care is essential in ensuring the durability and reliability of TransMotion Medical upholstery. In general, all products should be:

1. Cleaned
2. Disinfected (in accordance with facility policy)
3. Rinsed
4. Allowed to air dry

It is important to note that the terms “disinfecting” and “cleaning” should not be used interchangeably. Disinfectants alone will not provide adequate cleaning since they do not have the appropriate properties to cut grease or oil and remove grime, hair or skin oils. The appearance, feel and performance of your upholstery may diminish if not cleaned properly.

Cleaning:

Remove hair, grime and body oils. Your upholstery should be cleaned on a regular basis with a damp cloth soaked in a mild soap and water solution. Avoid harsh detergents or chemicals that could damage the finish of your product. If disinfecting with manufacturer approved chemicals or bleach, the upholstery MUST be wiped off using clean water on a damp cloth and allowed to air-dry as a final step. Failure to rinse upholstery with clean water can result in a build-up of residues that, over time, may lead to drying, cracking or other undesirable changes to appearance, feel and performance.

Retain all instructional tags for future use.

Contact the manufacturer of the fabric used in your upholstery for a current list of approved disinfectants. TransMotion Medical Customer Care can assist with identifying which upholstery was used with your Stretcher-Chair.

For customer supplied and non-standard materials, please refer to the individual manufacturer’s cleaning instructions.