



Field Installation and Usage Instructions

Part No. TMA200-15

Chart Rack

For all model TMM chairs except TMM3 Series



5516 S.W. 1st Lane – Ocala, FL 34474-9307
Phone: 1-800-237-3377 - Fax: 1-352-854-9544
www.transmotionmedical.com

TABLE OF CONTENTS

IMPORTANT NOTES ABOUT THIS MANUAL	3
SPECIAL NOTES - SIGNAL WORDS	3
TOOLS REQUIRED.....	4
PARTS INCLUDED FOR TMA200-15	4
PREPARING CHAIR FOR CHART RACK INSTALLATION	4
MOUNTING THE CHART RACK.....	5
CLEANING INSTRUCTIONS	6
PREVENTIVE MAINTENANCE	7

IMPORTANT NOTES ABOUT THIS MANUAL

This Field Installation and Usage Instruction manual is intended to assist you with installation of your TransMotion Medical Product. Carefully read this manual before using the equipment or doing service / maintenance on it. If you are unable to understand the WARNINGS, CAUTIONS, and instructions, contact TransMotion Medical Customer Service before attempting to operate or service the equipment. Otherwise, personal injury or property damage may result.

To ensure safe operation of this device, it is essential that methods and procedures be established for educating and training staff on the safe and effective operation of the product.

TransMotion Medical reserves the right to change specifications without notice.

The information contained in this document is subject to change without notice.

Specifications listed are nominal, and operation may vary slightly from unit to unit due to tolerance variations.

Figures contained within this document may not depict exactly the model you have. This document may cover many closely related models.

SPECIAL NOTES - SIGNAL WORDS

Signal words are used in this manual and apply to hazards which could result in injury or property damage. The following is a definition of those signal words as used in this document:

DANGER

Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices or potential property damage hazards.

NOTICE

Provides important information, makes special instructions clearer, or provides service personnel information to make maintenance easier.

TOOLS REQUIRED

Power Drill
1/4" (.250) drill bit
Flat screw driver
3/8" wrench
Marking utensil

PARTS INCLUDED FOR TMA200-15

- (1) Chart Rack
- (4) #10-24 x 1/2" long Truss Head Machine screws
- (4) #10-24 Lock Nuts
- (4) #10 Flat washers

If any features are missing or seem to be damaged due to shipping, contact TMM Service Department toll-free at 1-800-237-3377.

PREPARING CHAIR FOR CHART RACK INSTALLATION

- 1) Move the Stretcher-Chair into the seated position.
- 2) Lower the push bars
- 3) Remove the back / seat cushion.



Mounting the chart rack on a TMM4 Series chair will interfere with X-Rays.



MOUNTING THE CHART RACK

1) Position the chart rack on the back of the chair at desired location.

2) Using a marking utensil such as a marker or pen, mark the location of the (4) holes to be drilled in the back section.

3) Remove the Chart Rack and drill (4) ¼" (.250) diameter holes at the previously marked locations.

WARNING

Use proper personal protection when using power tools. Failure to do so could cause serious bodily injury.



4) Mount Chart Rack using (4) Truss head screws, (4) Flat Washers and (4) Lock nuts.



Truss Head screws must be used on the patient side of the chair back section.



Ensure Nuts are tight on Chart Rack. Failure to tighten nuts could cause Chart Rack to malfunction.

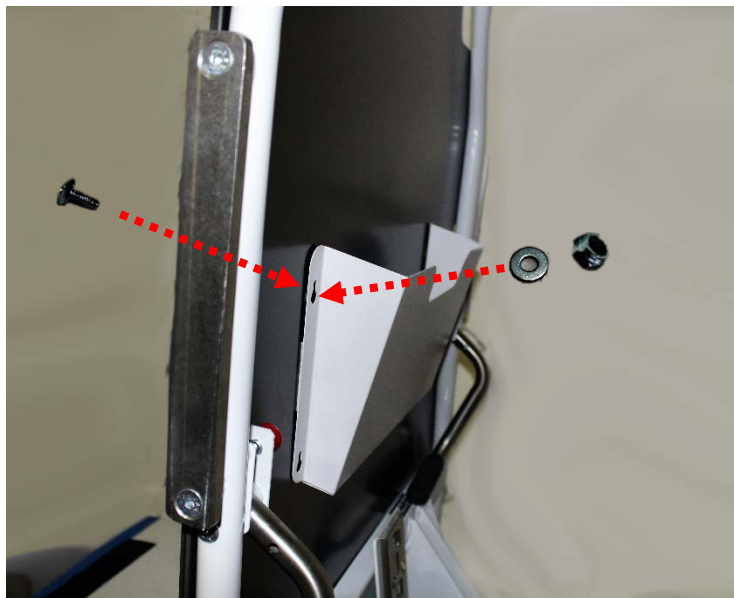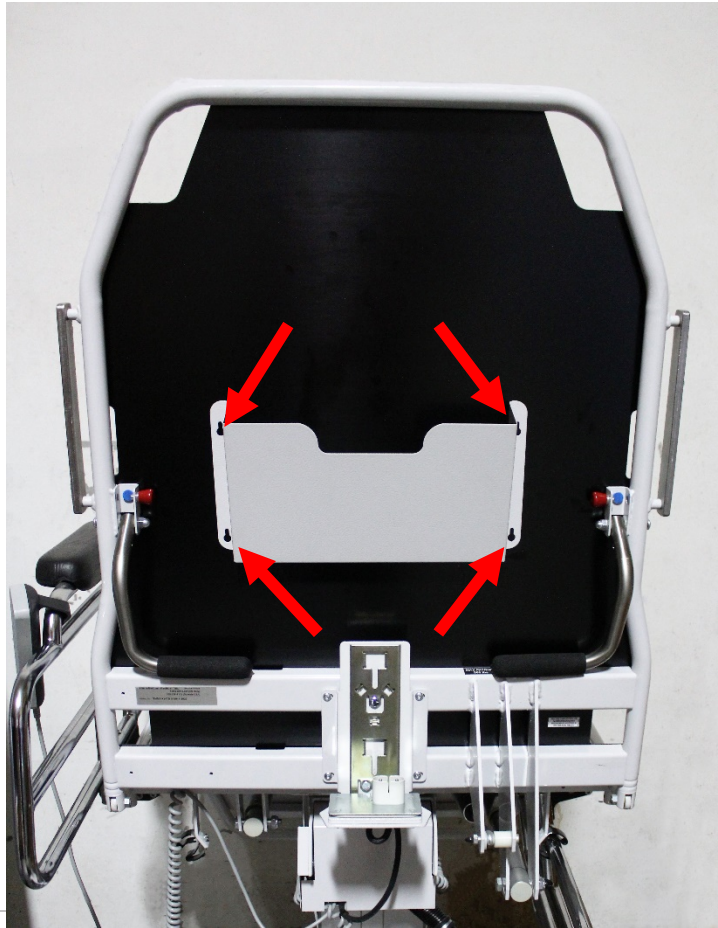

CLEANING INSTRUCTIONS



STEAM AND/OR PRESSURE CLEANING CHAIR WILL VOID WARRANTY

COMPONENT	FREQUENCY	PROCEDURE	CLEANING AGENT *	COMMENT
All Rails and Painted Metal	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for chipped paint/chrome
Base Cover	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for peeling or missing labels
Actuators and Battery Pack	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
Pendants	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Clean exterior surfaces only with minimal water
All Other Surfaces	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
TransMotion Medical Accessories	As required/in accordance with facility policies	Wipe with damp cloth and allow to air dry	Facility approved detergents, disinfectants & water	Check for damage
3rd Party Accessories	As required/in accordance with facility policies	Consult manual provided by accessory manufacturer.		

* Use of Facility approved detergents and disinfectants must comply with the instructions provided by the manufacturer(s) of those products.

Upholstery Care & Cleaning Instructions

Proper care is essential in ensuring the durability and reliability of TransMotion Medical upholstery. In general, all products should be:

1. Cleaned
2. Disinfected (in accordance with facility policy)
3. Rinsed
4. Allowed to air dry

It is important to note that the terms “disinfecting” and “cleaning” should not be used interchangeably. Disinfectants alone will not provide adequate cleaning since they do not have the appropriate properties to cut grease or oil and remove grime, hair or skin oils. The appearance, feel and performance of your upholstery may diminish if not cleaned properly.

Cleaning:

Remove hair, grime and body oils. Your upholstery should be cleaned on a regular basis with a damp cloth soaked in a mild soap and water solution. Avoid harsh detergents or chemicals that could damage the finish of your product. If disinfecting with manufacturer approved chemicals or bleach, the upholstery MUST be wiped off using clean water on a damp cloth and allowed to air-dry as a final step. Failure to rinse upholstery with clean water can result in a build-up of residues that, over time, may lead to drying, cracking or other undesirable changes to appearance, feel and performance.

Retain all instructional tags for future use.

Contact the manufacturer of the fabric used in your upholstery for a current list of approved disinfectants. TransMotion Medical Customer Care can assist with identifying which upholstery was used with your Stretcher-Chair.

For customer supplied and non-standard materials, please refer to the individual manufacturer's cleaning instructions.

PREVENTIVE MAINTENANCE

To ensure proper operation, this accessory should be inspected and maintained as described below.

COMPONENT	FREQUENCY	PROCEDURE	TOOL	COMMENT
Mounting Hardware	Every three months	Inspect mounting hardware to ensure that it is not coming loose.	3/8" wrench	-
Chart Rack	Every six months	Inspect Chart Rack for broken or loose welds. Inspect for chipped coating.	-	Replace chart Rack as required.