
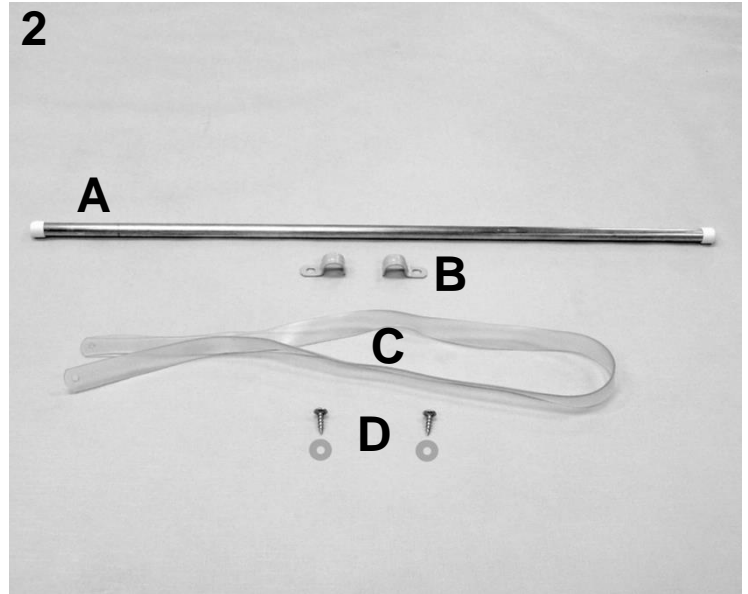


**PARTS LIST**

Image Number	Note	Part Number	Description	QTY per Product
1	A	<b>U3003XX</b>	<b>850 UPHOLSTERED TOP</b>	1
1A	B	222700	TABLE FRAME 850/851/855	2
1A	B	701010	1X2X18 RECT INSERT CAP/GLIDE LT.GRAY	4
1A	B	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
1A	B	566528	SCREW-TAP 10 X 2 3/4 PAN PHIL T/S ZINC	4
2		222720	850/851/855/856 WELDED BRACE ASSM.	1
2		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
3	C	222730	WELDED, 850-851-856 REINF. BRACE ASSM WELDED	1
3		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
3		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4	D	222731	WELDED, 850-851-856 REINF. BRACE ASSM WELDED	1
4		566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
4		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4

**NOTES**

Note	Description
A	XX- Use color code for specific part number i.e. 02 - Royal Blue
B	These parts are included with the above listed UPHOLSTERED TOPS
C	For tables with optional cabinet, or drawer. Must order double-sided tape listed on <b>Page 3</b>
D	For tables with optional shelf. Must order double-sided tape listed on <b>Page 3</b>

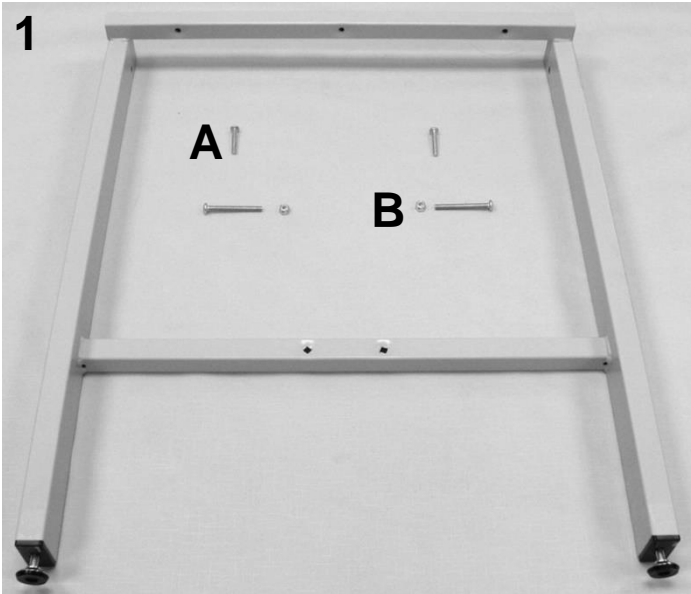

**PARTS LIST**

Image Number	Note	Part Number	Description	QTY per Product
1		<b>222708K</b>	<b>WELDED LEG ASSM. KIT 19"</b>	2
1	A	222708	WELDED LEG ASSM. 19"	1
1		<b>222709K</b>	<b>WELDED LEG ASSM. KIT 25" (850 style)</b>	2
1	A	222709	WELDED LEG ASSM. 25" (850 style)	1
1		<b>222710K</b>	<b>WELDED LEG ASSM. KIT 30" (850 style)</b>	2
1	A	222710	WELDED LEG ASSM. 30" (850 style)	1
1		<b>222711K</b>	<b>WELDED LEG ASSM. KIT 36" (850 style)</b>	2
1	A	222711	WELDED LEG ASSM. 36" (850 style)	1
1	A	500350	GLIDE NON-SWIVEL ZINC 5/16X1" NP LN	2
1	A	501065	1"x2" METAL GLIDE INSERT;18 GAUGE;5/16-18 THREAD	2
1A	A	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	2
1B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	2
1B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	2
2A		<b>222705K</b>	<b>PAPER ROD HOLDER ASSM. KIT (850 STYLE)</b>	1
2A	A	222705	850/851/855/856 ROD, PAPER HOLDER 1/2" ALUM	1
2A	A	700650	1/2" PLASTIC CAP, WHITE	2
2B		201810	PAPER ROD CLAMP, GRAY	2
2C		271500	VINYL TABLE STRAP	1
2D		566302	WASHER-#10 SAE F/W ZINC	2
2D		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	2
<b>NOTES</b>				
Note	Description			
A	These parts are included in ALL above listed kit.			



PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		<b>263960K</b>	<b>850 SERIES PLAIN SHELF KIT</b>	1
1	A	263960	850 SERIES PLAIN SHELF	1
N/A	A	705600	DOUBLE SIDED FOAM TAPE, 3/4" X 36YDS	1 ft.
2		263950	850,851,856 MODULE SHELF	1
3		<b>266780K</b>	<b>850/851/856 CABINET MODULE ASSM. KIT</b>	1
3A	A	266780	850/851/856 CABINET MODULE	1
3A	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
3A	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4		<b>266730K</b>	<b>850/851/856 DRAWER MODULE KIT</b>	1
4	A	266730	850/851/856 DRAWER MODULE	1
4B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
4B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
4A	A	<b>266760K</b>	<b>850/851/856 MODULE DRAWER ASSM. KIT</b>	1
4A	A	266760	850/851/856 MODULE DRAWER ASSM	1
4B	A	566519	SCREW-MACH 1/4-20 X 2 1/2 PAN PHILLIP M/S ZINC	4
4B	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
NOTES				
Note	Description			
A	These parts are included in the above listed kit.			



Revision History

REV	Effective Date	CCR Number	Description of Change
01	11/27/17	DCC 17-071	Misc. minor changes.