

NOTICE MAXIMUM PATIENT WEIGHT IS 500 lbs (227 kg.)

1. SIDE RAIL - LOWERING/RAISING



Lowering the Side Rail

Grasp the top of the side rail and push slightly inward while pulling out the Red release tab at the bottom of the rail.

Stowing the Side Rail

Lower the rail into the down position.



Raising the Side Rail

Lift the side rail until it is in the up position. The Red release tab will engage (lock) automatically.

NOTICE

After raising, pull on side rail to ensure it is securely locked.

2. BACK SECTION QUICK RELEASE



Location

The (Red) quick release lever is located under the seat on the patient's right side.

Activating the Quick Release

Pull the Red lever out towards the side rail. When weight is applied, the back section will drop until the lever is released.

NOTICE Intended for EMERGENCY purposes ONLY.

3. CASTER BRAKE OPERATION



Brake Mode

Activate the braking system by pressing down on the red end of either brake pedal.



This mode prevents all four casters from swiveling and all four wheels from spinning.



Neutral Mode

Activate the neutral mode by placing either brake pedal into a horizontal orientation.



This mode allows all four casters to swivel and all four wheels to spin freely.



Steer Mode

Activate the steer-locking system by pressing down on the green end of either brake pedal.



This mode locks front right caster parallel to the base but allows the other three casters to swivel and all four wheels to spin freely.

4. BATTERY PACK ("B" OPTION) INSTALLATION AND REMOVAL



Battery Pack Removal

Grasp the battery pack's integrated handle and lift up until locks disengage then pull the battery away from the mounting bracket.



Installing the Battery Pack

Place the two circular pins located on the back of the battery pack into the square openings on the battery mounting bracket. Once the pins are aligned, slide the battery pack down until the pins lock into place.



To maintain an optimum charge, Winco recommends charging your battery every 2-3 days.



5. PUSH BAR OPERATION



Lowering the Push Bar

Grasp the push bar handle while pulling out the Red release knob and lower the push bar.


Raising the Push Bar

Lift up on the push bar handle until the Red release knob engages (locks).


7. TEN BUTTON PENDANT

Pendant Storage

To prevent damage, the pendant may be stored at various locations on the chair when not in use.

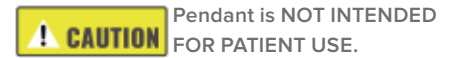

Pendant is NOT INTENDED FOR PATIENT USE.

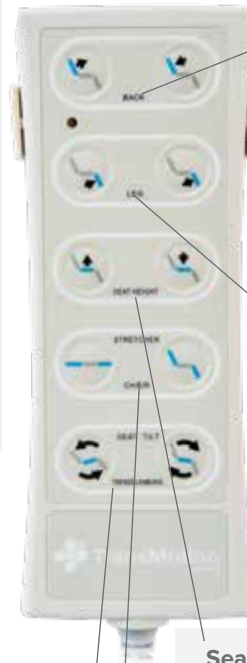

Back Adjustment

The first row of buttons controls the angle of the back section. Press the left button to raise the chair back. Press the right button to lower the chair back.

Leg Adjustment

The second row of buttons controls the angle of the leg section. Press the left button to raise the leg section. Press the right button to lower the leg section.

Seat Height Adjustment

The third row of buttons controls the seat height. Press the left button to raise the seat. Press the right button to lower the seat.

Auto-Contour Adjustment

The fourth row of buttons controls the simultaneous actuation of the leg and back sections. Press the left button to place the chair in a stretcher position. Press the right button to place the unit into the chair position.

Seat Tilt / Trendelenburg Adjustment

The fifth row of buttons controls the incline/recline of the seat section. Press the left button to tilt the seat backward. Press the right button to tilt the seat forward.



PATIENT MUST ENTER / EXIT FROM THE SIDE WITH THEIR BODY WEIGHT CENTERED OVER SEAT SECTION.

PATIENT INGRESS / EGRESS SHOULD ALWAYS BE MADE WITH THE UNIT IN AN UPRIGHT-CHAIR POSITION WITH CASTERS LOCKED.



For additional assistance, please feel free to contact us at:

800-237-3377 | WincoMfg.com | 5516 SW 1st Lane, Ocala, FL 34474 | techservice@WincoMfg.com

Copyright 2019 Winco Mfg., LLC | TMM-2215-13 | REV A